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CHILDREN IN WALKER “KANGAROO” FEEL FREE AND EXPERIENCE FEELINGS OF SELF-WALKING. FREQUENT USE OF WALKER “KANGAROO” HELPS TO GET WALKING HABITS. LATER, THE AID CAN BE USED FOR SKIING OR SKATING (SUITABLE ALSO FOR SEVERELY DISABLED CHILDREN).

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BAZALSNUZ® MAXIM-ZDR.CZ®

Creative workshop MAXIM has been on the Czech market for ten years. Arose from the needs of mothers of physically and mentally debilitated children who helped with the development of individual utilities. Each product is linked to the story of a particular person. Each product helped him in his daily life.

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KANGAROO A VERTICAL AID FOR PRACTICING WALKING

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IS A VERTICAL AID FOR PRACTICING WALKING FOR SEVERELY PHYSICALLY DISABLED CHILDREN

It consists of two separate parts. Panties - part for the child is in the form of diaper supplemented straps with button-clasp and secure fastening to the abdomen. Part of the accompaniment consists of a fixed part for his back, the straps are held to the accompaniment so he can simply turn on the straps into the button-clasp that are on the child's panties. Child's weight is transferred back to accompaniment.

Secure suspension of the child frees up hands of accompaniment and allows doing other tasks. Movements of the pelvis when walking are transmitted from accompaniment to a child that instinctively transfers weight from right foot to the left. When we pull the lower straps on the back of the child panties (the child is close to the accompaniment) the copying movements of the child pelvis will increase. Very important is a solid set of the straps child. Using "eye of the button-clasp" can change the settings.

Children in walker "Kangaroo" feel free and experience feelings of self-walking. Frequent use of walker "Kangaroo" helps to get walking habits. Later, the aid can be used for skiing or skating (suitable also for severely disabled children).

We recommend consulting with your doctor. If the child's hip is alright doctor the aid „Kangaroo“ normally approve. With the "Kangaroo" children may also submit four, by switching carabines.

Another use option is to "hang a child in free space", for example in the kitchen when mother is cooking or in the playroom with other children.

There are three sizes according to height and waist circumference of the child.

SIZE:

I. From the height 90cm	Waist 55-60cm	Product Code 7,0021
II. From the height 110cm	Waist 60-70cm	Product Code 7,0022
III. From the height 130cm	Waist 70-80cm	Product Code 7,0023

MATERIAL: 100% cotton, foam padding, straps, Velcro, button-clasp.

WEIGHT: For all sizes is the load capacity up to 75 kg. The child must be a head shorter than the accompaniment.

MAINTENANCE: Washable in washing machine at 40° - 60°C.

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